

# Equipment Maintenance Log

PROFESSIONAL  
PROTECTOR PLAN®  
FOR DENTISTS



Equipment	Daily	Weekly	Every Other Week	Monthly	Every Other Month	Every Six Months	Yearly	As Needed
Sterilizer, Statim								
- Empty used water container								XX
- Lubricate cassette seal			XX					
- Apply Stat Dry to cassette			XX					
- Replace air filter						XX		
Sterilizer, Pelton								
- Clean chamber					XX			
- Spore test					XX			
- Check door seals					XX			
- Activate safety valve						XX		
Chairs								
- Clean		XX						
Operatory light								
- Clean shield		XX						
- Check for drift				XX				
- Clean reflector								XX
Dental Unit								
- Clean out vacuum trap		XX						
- Flush handpiece tubings	XX							
- Clean vacuum lines			XX					
Nitrous Oxide System								
- Turn system on/off	XX							
- Check rubber components for wear					XX			
- Perform overnight pressure test						XX		
Automatic Film Processor								
- Check chemicals	XX							
- Clean racks		XX						
- Change chemicals				XX				
- Clean entire unit					XX			

*Any publications or forms on this website are for informational and educational purposes only. Nothing contained within this website or on any publications or forms found therein is intended to be legal or dental advice. Accordingly, PPP makes no representations regarding the correctness or completeness of the aforementioned content and accepts no liability for any injury or damage that may arise from its use by persons viewing this website. Any person viewing this website should direct any specific legal or dental questions to a competent attorney or dental professional. In addition, the information contained within this website or on any publications or forms found therein may contain or refer to matters which are outside the scope of your insurance policy, and such information and materials do not create or imply the existence of coverage. Every insured should consult its insurance policy for the specific terms and conditions of coverage.*

# Equipment Maintenance Log

PROFESSIONAL  
PROTECTOR PLAN®  
FOR DENTISTS



Equipment	Daily	Weekly	Every Other Week	Monthly	Every Other Month	Every Six Months	Yearly	As Needed
Compressor								
- Check oil level		XX						
- Check moisture indicator			XX					
- Check air intake filters				XX				
- Change oil							XX	
- Regenerate desiccant								XX
Vacuum Pump								
- Check and clean filter		XX						
- Check for water leaks		XX						
- Run cleaner through lines		XX						
X-ray Heads, Intraoral								
- Check arms for drift				XX				
- Check tube head for oil leaks				XX				
- Verify audible alert				XX				
- Check lead aprons				XX				
- Calibrate settings								2 years
X-ray, Panoramic								
- Verify audible alert				XX				
- Check lead aprons				XX				
- Check cassettes					XX			
- Check cassette holder					XX			
- Calibrate settings								2 years

*Any publications or forms on this website are for informational and educational purposes only. Nothing contained within this website or on any publications or forms found therein is intended to be legal or dental advice. Accordingly, PPP makes no representations regarding the correctness or completeness of the aforementioned content and accepts no liability for any injury or damage that may arise from its use by persons viewing this website. Any person viewing this website should direct any specific legal or dental questions to a competent attorney or dental professional. In addition, the information contained within this website or on any publications or forms found therein may contain or refer to matters which are outside the scope of your insurance policy, and such information and materials do not create or imply the existence of coverage. Every insured should consult its insurance policy for the specific terms and conditions of coverage.*